

Cubro Easireach Stick

Providing you the ability to pick things up easily after surgery.

Use

Pull the trigger at the handle end to operate the grab arms at the other end.

Features

- Enhanced grip.
- Magnetic end.
- Clip for walking stick or frame.

Features:

- With clip for walking stick.
- Frame and magnetic end.
- Lightweight.
- Smooth and positive clamping action.
- With semi circles cut into the top jaw.

How to adjust length

1. Loosen anti-rattle collar/knob.
2. Depress spring button.
3. With arm down at your side, adjust cane length to point where wrist bends.
4. Slide extension up or down until spring button protrudes through an adjustment hole.
5. Hole located on handle end of cane.
6. You will hear an audible "click" when spring button securely protrudes through the adjustment hole.
7. Retighten anti-rattle collar/knob.

How to care for and maintain quality?

1. Make sure that the spring button is released and re-engaged properly.
2. Inspect the cane tips often for any wear and damage.
3. Replace any broken, damaged or worn parts immediately to ensure safety of use.